QUICK GUIDE

AIRFLOW® PROPHYLAXIS MASTER

"I FEEL GOOD"

GUIDED
BIOFILM
THERAPY®

EMS+
MAKE ME SMILE.

DOWNLOAD ON MYEMS.DENTAL


General Instructions:
installation, cleaning,
reprocessing, maintenance ...

Treatment Recommendations
Guided Biofilm Therapy

Extended use
of PIEZON®
instruments for
Endodontology
and other
dental
applications.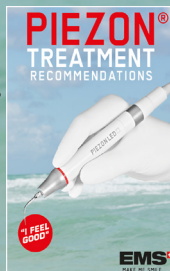

EMS Dental
Installation & Maintenance

SUMMARY

EN ENGLISH	p.4	LV LATVIEŠU	p.64
BG БЪЛГАРСКИ	p.8	MS BAHASA MELAYU	p.68
CS ČESKÝ	p.12	NL NEDERLANDS	p.72
DA DANSK	p.16	NO NORSK	p.76
DE DEUTSCH	p.20	PL POLSKI	p.80
EL ΕΛΛΗΝΙΚΑ	p.24	PT PORTUGUÊS	p.84
ES ESPAÑOL	p.28	RO ROMANIAN	p.88
ET EESTI KEEL	p.32	RU РУССКИЙ	p.92
FI SUOMI	p.36	SK SLOVENSKY	p.96
FR FRANÇAIS	p.40	SL SLOVENŠČINA	p.100
HR HRVATSKA	p.44	SV SVENSKA	p.104
HU MAGYAR	p.48	TH ไทย	p.108
ID INDONESIA	p.52	TR TÜRKÇE	p.112
IT ITALIANO	p.56	VI TIẾNG VIỆT	p.116
LT LIETUVIŠKAI	p.60	ZH 英文	p.120

WELCOME

We thank you for your trust in our newest product AIRFLOW® Prophylaxis Master, which will guide you into the world of our GBT - GUIDED BIOFILM THERAPY.

Register your new unit on www.myems.dental and get an extra year of warranty. This warranty is only valid if you have completed your first yearly maintenance within 12 months following purchase.


Before installation, download & read complete instructions for use (myems.dental).


Connections

- + EH-142 Air hose to the cabinet / dental unit.
- + EG-110 Water hose to the cabinet / dental unit.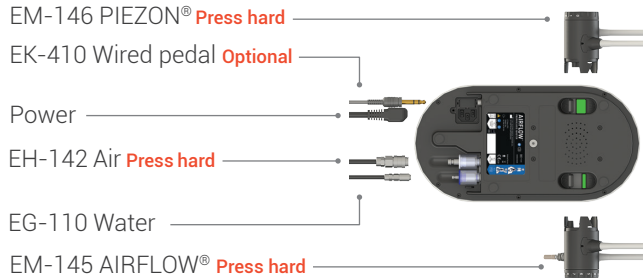

Check cords are locked.

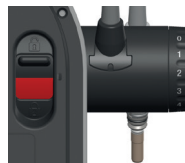Install the batteries in the pedal.

AIRFLOW PLUS: Sub and Supragingival.

AIRFLOW COMFORT: Supragingival on enamel only.

Insert the powder chamber.


The use of the PIEZON® Bottle is optional.

Functions

If the bottle is not placed onto the device then you are connected to the external water supply.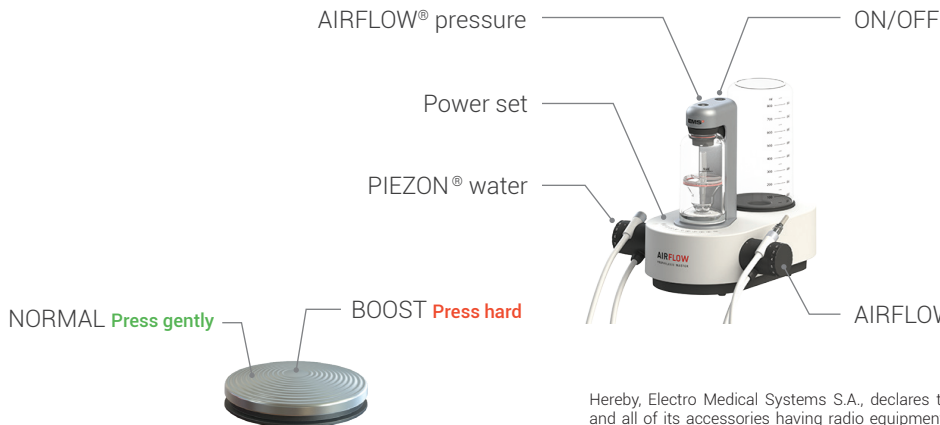

Hereby, Electro Medical Systems S.A., declares that this Medical Device and all of its accessories having radio equipment are compliant with the European Directive 2014/53/EU (RED Radio Equipment Directive). The full text of the EU declaration of conformity is available in the Instruction for Use.

CLEAN

EN

EACH MORNING Rinse

①

Place your water bottle onto device

②


Set water to 10, on both sides

③

Clip the 2 cords on the CLIP+CLEAN over a sink

④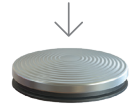

Press the pedal once, wait for ~1 minute

BETWEEN Each patient

①

Clean outside surface of device with a cloth and alcohol

②

Reprocess your Handpieces and instruments

END OF DAY Clean

①

Place your blue bottle onto device, filled with NIGHT CLEANER

②


Set water to 10, on both sides.

③

Clip the 2 cords on the CLIP+CLEAN over a sink.

④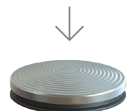

Press the pedal once, wait for ~1 minute.

MAINTENANCE


CHECK IT

Check the blue water filter and change it if needed.

Renew your PIEZON® light guide if needed.

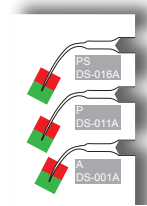Check instrument wear. See back cover of your guide.

EACH YEAR Preventive maintenance

Send your device to your authorized EMS repair center.


TROUBLESHOOTING


Handpiece clogged


Use Easy Clean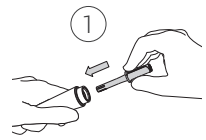

Place the Easy Clean into the handpiece.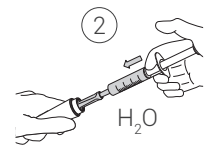


Then take a syringe filled with water.

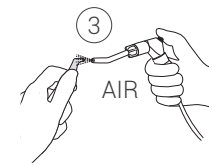Dry both sides of handpiece with compressed air.

Service needed
Stop using your device.

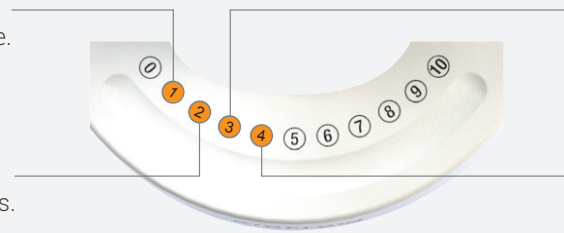Batteries low
Change pedal batteries.

Missing cord
Check your cords connection.

Overheating
Wait for cooling.
If error repeats, call service.

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AU

SYDNEY, AUSTRALIA
E.M.S. ELECTRO MEDICAL SYSTEMS OCEANIA
 Unit 1, Building 4 – Sydney Corporate Park
 160 Bourke Road
 AU - Alexandria, NSW 2015
 T. +61 405 095 867
 E-mail: info@ems-oceania.com


JP

TOKYO, JAPAN
E.M.S. JAPAN BRANCH OFFICE
 Eminent Ningyoucho Bld 6F,
 9-18, Kobuna-cho, Nihombashi,
 JP - Chuo-ku, Tokyo, 103-0024
 T. +81 (0) 3-6810-9351
 F. +81 (0) 3-6810-9352
 E-mail: emsjapan@ems-ch.com

CN

SHANGHAI, CHINA
医迈斯电子医疗系统贸易(上海)有限公司
E.M.S. ELECTRO MEDICAL SYSTEMS TRADING (SHANGHAI) CO., LTD.
 24A, Jin Sui Mansion,
 N°379 Pudong Nan Rd.
 CN - Shanghai 200120
 Tel. +86 21 3363 2323
 E-mail: emschina@ems-ch.com

FOLLOW US

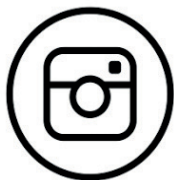EMS Dental



EMS Dental -
International

EMS Dental

ONLY USE EMS ORIGINALS

Check and renew regularly your
PIEZON® Instrument.

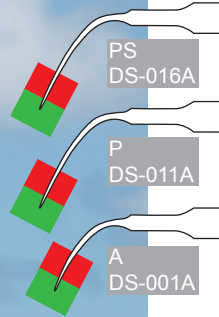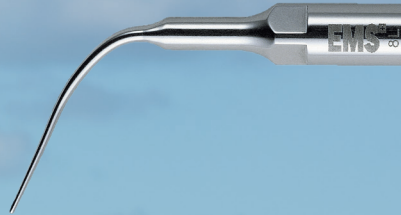Only use original AIRFLOW® Powders

Beware of so-called „EMS compatible“ powders in the market, which can damage oral tissues and the device. See here a nozzle completely damaged by such aggressive powders.

- ▶ Use only EMS powders with your EMS devices.
- ▶ The use of not authorized powders by EMS will result in a loss of warranty.

CE 0124

EMS 

PRINTED IN SWITZERLAND
WWW.EMS-DENTAL.COM